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---

EN

Dental Wings has released its latest upgrade of the DWOS software. Version 7.2 is compatible with the following Dental Wings scanners: 7Series, 3Series, and iSeries, as well as standalone software licenses. This version is not compatible with 5Series scanners.

The following is the complete list of new features and improvements:

## New Features

### Inbox / DWOS Connect

- Cases sent from the Dental Wings Intraoral Scanner without a treatment plan can now be imported as a Bite Splint, Partial, Orthodontic, or Model Builder order.
- New connection type - A DWOS Lab connection is now available to automatically retrieve new orders from a lab management instance, immediately ready for scan or design in CAD software, etc.

### Other

- 3D mouse from 3Dconnexion official driver support - Official drivers from 3Dconnexion within the application are now available. Different options from the driver tool may be customized, and any product model may be used.

## Improvements

### Crown & Bridge / Implant

- Readapt anatomy - New 3D preview of the selected anatomy provides more details on the selected anatomy.
- Possibility to show/hide the gingiva scan when designing a margin line on a stump.
- Spacers are now available for combined orders (Crown & Bridge mixed with partial case).
- The selection of implant kits in DWOS Synergy workflow is made easier with a new dialog.

### Virtual wax-up

- Adaptation of anatomy on diagnostic wax-up.
- Mirror anatomy now available with Virtual Wax-up orders.

## Model Builder

- Dreve Socket - renamed to "Flexible" and allows now to print or mill locally
- Control window with attachment enhanced to make hole in the stump, model, and gingiva.
- Migration of analog and articulator parameters inside the material definition.
- Import of external file (STL) available in Model Builder station.
- Improved user interface to differentiate the type of models available.
- Improved stability when two analog sockets intersect.

## Partial Framework

- Improved: Inferior bars of rounded profile now respect user's required dimensions.

## Other

- Improved the global stability of the software.

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