

Release Notes DWOS[®] 8.1.4

December 2018

Dental Wings has released a new update for the DWOS dental CAD/CAM software. DWOS 8.1.4 is compatible with the Dental Wings 7Series, 3Series, and iSeries desktop scanners, as well as standalone software licenses.

Here is the complete list of bug fixes:

Arch Scan

- Fixed an issue that prevented the implant repositioning verification.

Crown & Bridge / Implant

- The placement of **Locator** attachments on an implant bar is made easier.

Model Builder

- Added missing .pol files into **Witlaken** and **Zimo** output packages.

Bite Splint

- The **Remove Interference** function with interferences computed from the virtual articulator has been fixed.

Others

- Fixed antagonist display when using **Route Back** on an order from CAD to scan step.
- Made SUM3D output format available for virtual wax-up orders.
- Fixed an error that occurred when using the in-house scan import workflow.