

Release Notes v. 2023.2*

What we've fixed

We fixed the bugs below to make your CAD experience better.

Order downloading

Orders can now be downloaded from 3Shape Communicate in Inbox.

Chinese language

The DWOS interface is now available in Chinese.

Order Management

Virtual waxups are calculated more quickly than ever.

Save vertical axis adjustment

Replace scan files in the scan import station and save the model's vertical axis adjustment.

Scan import station

Save scan files when replacing with other scan files in the scan import station.

What we've improved

We're doing our best to improve DWOS every day.

Increased post design height

Abutment post designs now have a prescribed height of at least 4 mm.

Increased security

On systems running Windows 7 the cryptographic TLS protocol has been updated to increase protection against cybersecurity threats.

*This version is compatible with Dental Wings standalone software licenses and the following lab scanners:
7Series, 3Series, and Medit T500, T310, T510, and T710.